



APPLICATION

The firing pad is applied directly from the syringe into the crown or applied onto the pressed ceramic object, then carefully placed on the pin of the firing tray.

FIRING

In the ceramic furnace, the special material completely hardens and thereby stabilises the technician's work during the firing process – a secure support for inlays, onlays crowns and veneers.

Errors and printing errors excepted. Subject to change at any time.

Technical inquiries: technic@creation-willigeller.com

REMOVAL

After firing, the solidified paste can be quickly removed under running water with an instrument, without leaving any residue. If necessary, it can finally be blasted off with polishing beads under a maximum of 2 bar of pressure.

When using Creation LF veneer materials, it is absolutely essential to avoid contact with the firing pad.

STORAGE

The liquid firing pad should be stored in a cool, dry place, maintaining an airtight seal.

Distributor

Creation Willi Geller International GmbH
Koblacherstraße 3, 6812 Meiningen, Austria
Tel. +43 (0)5522 76784
Fax. +43 (0)5522 73699
info@creation-willigeller.com
www.creation-willigeller.com

Manufacturer

KLEMA Dentalprodukte GmbH
Koblacherstr. 3a, 6812 Meiningen, Austria

