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PLEASE READ CAREFULLY BEFORE USE. KEEP FOR REFERRAL PURPOSES.

**EN**

## CREATION VC Composite Primer

Primer for micro-filled composite and acrylic resin. For use only by a dental technician in the recommended indications.

### INDICATIONS

- Surface treatment of micro-filled resin (MFR) or micro-filled hybrid resin (CREATION VC Veneering Composite, CREATION VC Gingiva) prior to luting or repairing procedures.
- Surface treatment of acrylic resin prior to repairing procedures.

### CONTRAINDICATIONS

This product must not be used for patients with known allergies to methyl acrylate monomer or methyl acrylate polymer.

### USE

#### A. Surface treatment prior to luting

- Preparation

Prepare the surface to be bonded using a sandblaster (aluminium oxide) with a pressure of at least 3 kg/cm² (2,94 bar). Observe to the manufacturer's instructions for use. Clean with oil-free air.

**Note:** mask surfaces not to be treated.
- Application

Dispense one or two drops of CREATION VC Composite Primer into a dispensing dish. Apply a thin layer to the surfaces to be bonded using a brush.

**Note:**

  - Use immediately after dispensing. Close the bottle immediately after use.
  - Should the applied material be contaminated prior to light curing, wash and dry the surface and repeat the procedure by re-applying material.
- Light curing

Light cure using a visible light curing unit (wavelength 470 nm). Observe the manufacturer's instructions for use. After light curing, do not remove the oxygen inhibiting outer layer.

Halogen/LED (> 700mW/cm²): 20 seconds

High power LED (> 1200mW/cm²): 10 seconds

GC STEPLIGHT SLI-I\* : 10 seconds

GC LABOLIGHT DUO\* : 60 seconds

Plasma arc: 3 seconds
- Cementation

Apply a suitable luting cement on the treated surfaces of the restoration and continue with the luting procedure. Observe the manufacturer's instructions for use.

#### B. Surface treatment prior to reparations

- Preparation

Prepare the surface to be bonded using a diamond bur or a sandblaster (aluminium oxide) with a pressure of at least 3 kg/cm² (2,94 bar).
- Application

Dispense one or two drops of CREATION VC Composite Primer into a dispensing dish. Apply a thin layer to the surfaces to be bonded using a brush.

**Note:**

  - Use immediately after dispensing. Close the bottle immediately after use.
  - Should the applied material be contaminated prior to light curing, wash and dry the surface and repeat the procedure by re-applying material.
- Light curing

Refer to A.3. When using GC LABOLIGHT LV-III\*, light cure for 1 minute.
- Repair

Apply composite material such as CREATION VC Veneering Composite, CREATION VC Gingiva, CREATION VC Flow Dentine or CREATION VC Make up Neo and finish. Observe the manufacturer's instructions for use.

### STORAGE

Recommended for optimal performance: store in a cool, dark place (4-25 °C).

### SIZE OF PACKAGING

Bottle, 3 ml

### CAUTION

- CREATION VC Composite Primer is flammable. Do not use near open flame. Keep away from sources of ignition. Do not store large quantities in one area. Keep away from direct sunlight.
- In case of contact with oral tissues, immediately remove with a sponge or cotton pellet and flush oral tissue with water.
- In case of contact with skin, immediately remove with a sponge or cotton pellet. Rinse with soap and water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Absolutely avoid swallowing of the material.
- Do not mix with any other primers. Do not reuse a brush with other products.
- Always wear personal protective equipment (PPE) such as gloves, face masks and safety eyewear.
- In rare cases the product may cause cutaneous reaction.
- Seek medical attention.

\* Observe the manufacturer's instructions for use and make sure that these devices have the same light output as the GC LABOLIGHT LV-II, III, GC LABOLIGHT DUO and GC STEPLIGHT SL-I.

Some products referenced in this IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: https://www.creation-willigeller.com/en/media-center/manuals-broschures/

Undesired effects - Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts\_en as well as to our internal vigilance system: vigilance@gc.dental. In this way you will contribute to improve the safety of this product.

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Distributed by: GC AUSTRALASIA DENTAL PTY. LTD., 1753 Botany Rd, Banksmeadow, NSW 2019, Australia, Tel. +61 2 9301 8200