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PLEASE READ CAREFULLY BEFORE USE. KEEP FOR REFERRAL PURPOSES.	EN
---	-----------

CREATION VC Metal Primer

Primer for bonding resin to metal.

CREATION VC Metal Primer contains a special monomer (MEPS: thiophosphoric methacrylate) which promotes bonding of acrylic resins and composites to all types of metal.

For use only by a dental technician in the recommended indications.

INDICATIONS

Surface treatment of micro-filled composite (MFR) or micro-filled hybrid composite (CREATION VC Veneering Composite, CREATION VC Gingiva) prior to attachment with metals of all types.

CONTRAINDICATIONS

This product must not be used for patients with known allergies to methyl acrylate monomer or methyl acrylate polymer.

USE

A. GENERAL

- In all cases
 - Sandblast metal surfaces with aluminium oxide and clean with air gun.
Note: avoid contamination with oil, separator or moisture from fingers, etc.
 - Dispense 2-3 drops of CREATION VC Metal Primer into a dappen dish or similar container. Using the brush included in the delivery apply a thin layer to the bonding surface. Allow to dry.
Note: a second layer may be applied if necessary.
 - Apply resin to metal as described below.
Note: apply resin to metal immediately after preparation of the surface.

B. Resin cement to metal for metal inlays, crowns and bridges

Prepare metal surface as above and bond restoration with resin cement.

C. Opaque resin to metal for composite resin veneers and crowns

Prepare surface as above and apply opaque (CREATION VC Veneering Composite) in usual manner. Cure as recommended by the manufacturer. Observe instructions for use of the CREATION VC products.

D. Denture base to magnetic attachment

Prepare the surface as above and attach the attachment in the usual manner, e.g. with self-curing resin.

E. Denture base resin to metal base, clasps or attachments

- Sandblast bonding surface with aluminium oxide and clean with air gun.
- Form a wax denture.
- After burning out the wax, clean the bonding surface using a steam cleaner. Completely remove wax using a solvent.
- Apply CREATION VC Metal Primer.
- Pack and cure denture base in usual manner.

F. Repair of dentures with metal base, clasps or attachments

- Roughen metal bonding surface using carborundum point.
- Prepare bonding surface as above.
- Build up self-curing resin in the usual manner.

G. Bonding resin to metal orthodontic appliances

- Prepare bonding surfaces as above.
- Apply orthodontic resin to the appliance in the usual manner.

STORAGE

Recommended for optimal performance: store in a cool place (4-25 °C) away from direct sunlight.

SIZE OF PACKAGING

Bottle, 5 ml

Brush No. 7

CAUTION

- Contains methacrylate. Not to be used by operators, or on patients, with a history of allergy to methacrylates. In cases of allergic response, seek medical attention.
- In case of contact with skin or eyes, immediately flush with water and seek medical attention.
- Flammable. Avoid high temperatures, direct sunlight and sources of ignition. Use only in well ventilated rooms.
- Volatile. Seal container immediately after use.
- Use the brush (no. 7) included in the delivery ONLY for applying CREATION VC Metal Primer.
- Always wear personal protective equipment (PPE) such as gloves, face masks and safety eyewear.

Some products referenced in this IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: https://www.creation-willigeller.com/en/media-center/manuals-brochures/

Undesired effects - Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental. In this way you will contribute to improve the safety of this product.

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