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PLEASE READ CAREFULLY BEFORE USE.  
KEEP FOR REFERRAL PURPOSES.

EN

## CREATION VC Veneering Composite

Light-curing composite for producing crowns and bridges, inlays, onlays and veneers as well as for veneering crowns and bridges.

For use only by a dental technician in the recommended indications.

### INDICATIONS

- Veneering of crowns and bridges with metal framework.
- Metal free restorations: crowns, inlays, onlays and veneers.
- Implant suprastructures.

### CONTRAINDICATIONS

- Avoid use of this product in patients with known allergies to methacrylate monomer, methacrylate polymer or alcohol.
- In case of malocclusion and bruxism.

### USE

#### I. BUILDING UP PROCEDURE

##### A. Anterior/posterior veneer crown (with metal framework)

- Production of metal framework
  - Wax-up framework conventionally, minimum thickness of occlusal surface at least 1 mm. Retention beads can be added to the areas to be covered by resin.
  - Spruing, investing, casting. Remove retention beads in 0.5 mm width around the cervical margin.
  - Clean and sandblast with 50 µm aluminium oxide.
  - Dry and clean with air gun. Do not clean with steam.
  - Immediately apply with the brush a coat of CREATION VC Metal Primer to retention areas. Allow to dry.
- Build up veneer
  - Using the flat brush, immediately apply a layer of CREATION VC Base Opaque on the retention area. Light cure for 1 minute with GC LABOLIGHT LV-III\*.  
**Note:** for irradiation time with the GC visible light curing unit, refer to the chart given below.
  - Using the round brush, apply CREATION VC Cervical Opaque in 1 mm width along the cervical margin. Light cure for 1 minute.
  - Using the flat brush, apply two thin layers of CREATION VC Opaque to the entire surface. Light cure each layer for 1 minute.
  - Progressively apply CREATION VC Opaque Dentine as a cervical colour in 2-3 mm width around the cervical area. Light cure for 30 seconds.
  - Apply CREATION VC Dentine and Enamel, building up to the desired thickness. Light cure for 30 seconds. Use CREATION VC Make up Neo if required and light cure for 30 seconds.
  - If required, CREATION VC Neck Transpa can be applied to the cervical areas and CREATION VC Effect Enamel to the incisal edge. Light cure each layer for 30 seconds.
  - Coat the composite surface with CREATION VC Air Protect to ensure full polymerization of the surface and to avoid an oxygen inhibition layer.
  - Finally, light cure with the GC LABOLIGHT LV-III\* or GC LABOLIGHT DUO\* for 3 minutes. After light curing, remove CREATION VC Air Protect with water.
  - Grind and polish in usual manner. High-gloss finishing using CREATION VC Diamond Polisher on a felt wheel or leather buffing. **Note:** before polishing, clean the restoration well to completely remove the reaction layer, e.g. using alcohol. Attention: after using alcohol, control the surface for microscopic cracks and adjust as required.

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- Preparation of the working stone model
  - Prepare the stone die in usual manner.
  - Block out undercut with wax.
  - If required, apply a layer of paraffin wax as die spacer.
  - Apply a thin layer of CREATION VC Separator as resin composite separator.
- Special pre-cure treatment
  - In case of a metal core, apply first a thin layer of CREATION VC Cervical Opaque or CREATION VC Opaque. Light cure for 1 minute.
- Build up of crown
  - Apply one layer each of CREATION VC Opaque to the labial surface of the die and CREATION VC Cervical Opaque to the lingual surface of the stone. Light cure each layer for 1 minute.
  - Apply CREATION VC Opaque Dentine or Dentine (one shade number just below the crown shade) in the cervical areas. Light cure for 30 seconds.
  - For following build up of the labial surface, see instructions build up procedure I. A. 2. d-f.
  - Apply CREATION VC Opaque Dentine in the cervical area to cover 1/3 of the crown. Light cure for 30 seconds.
  - Progressively apply CREATION VC Enamel from the incisal edge towards the cervical area. Also apply CREATION VC Effect Enamel to the proximal surface.
  - Continue building up procedure A. (Section 2. g-i).

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- Preparation of the working stone model  
Follow the instructions preparation of the working stone model I. B. 1. a-d.
- Special pre-cure treatment  
Apply first a thin layer of CREATION VC Make up Neo to the cavity floor. Light cure for 30 seconds.
- Build up of inlay
  - Build up CREATION VC Neck Transpa as a dentine colour in order to retain the natural tooth colour. Light cure for 30 seconds.
  - In the case of a stained tooth, it is recommended to first apply CREATION VC Cervical Opaque, Opaque or Opaque Dentine to the cavity floor. Light cure CREATION VC Opaque and Cervical Opaque for 1 minute or CREATION VC Opaque Dentine for 30 seconds.
  - Build up CREATION VC Effect Enamel or CREATION VC Enamel, shape and contour. Light cure for 30 seconds.
  - Continue build up procedure according to I. A. 2. g-i.

##### II. ADDITIONAL APPLICATION AND REPAIR

- Roughen the resin surface using a carbide bur.
- Apply a thin layer of CREATION VC Composite Primer to improve adhesion. Light cure for 1 minute with the GC LABOLIGHT LV-III\* or GC LABOLIGHT DUO\*.
- Apply CREATION VC Effect Enamel, Enamel or Transpa, etc. as required.
- Coat the added resin surface with CREATION VC Air Protect.
- Light cure for 3 minutes.

P LEASE READ CAREFULLY BEFORE USE.  
KEEP FOR REFERRAL PURPOSES.

EN

### IRRADIATION TIME

Light curing unit	GC LABOLIGHT LV-III, II*	GC STEPLIGHT SL-I*	GC LABOLIGHT DUO*		
	Pre-cure	Final cure	Pre-cure	Pre-cure	Final cure
Paste					
Base Opaque Opaque Cervical Opaque	----	1 min.	----	----	1 min.
Dentine Opaque Dentine Enamel Effect Enamel Pearl Enamel Transpa Neck Transpa	30 sec.	3 min.	10 sec.	10 sec.	3 min.
Make up Neo	10 sek.	5 min.	20 sek.	10 sek.	1,5 min.
Flow Dentine	30 sek.	3 min.	10 sek.	10 sek.	3 min.

**Attention:** The pre-curing time stands for each layer and each single crown/pontic.

### STORAGE

Recommended for optimal performance:  
store in a cool place away from direct sunlight.

### SIZE OF PACKAGING

- Set Packages
  - CREATION VC Kit 1 (16 shades A1-D4), 54 syringes with accessories
  - CREATION VC Kit 2 (6 shades A1-A3.5, B2, B3), 30 syringes with accessories
  - CREATION VC Gingiva Kit (15 shades), 15 syringes with accessories
  - CREATION VC Make up Neo (19 shades), 2 syringes, 17 bottles with accessories
  - CREATION VC Accessory-Kit (content see 3. accessories)
- Refill Packages
  - CREATION VC Base Opaque, 2,4 ml (1 shade)
  - CREATION VC Opaque, 2,4 ml (16 shades, A1-D4)
  - CREATION VC Cervical Opaque, 2,4 ml (1 shade)
  - CREATION VC Opaque Dentine, 2,9 ml (5 shades, havanna, curry, orange, ivory, cuba)
  - CREATION VC Dentine, 2,9 ml (16 shades, A1-D4)
  - CREATION VC Enamel, 2,9 ml (4 shades)
  - CREATION VC Pearl Enamel, 2,9 ml (2 shades, blue, flamingo)
  - CREATION VC Effect Enamel, 2,9 ml (3 shades, light yellow, medium yellow, heavy yellow)
  - CREATION VC Transpa, 2,9 ml (4 shades, blue, white, yellow, grey)
  - CREATION VC Neck Transpa, 2,9 ml (3 shades, khaki, sand, honey)
  - CREATION VC Make up Neo, 2,6 ml (17 shades, MUN-A, MUN-B, MUN-C, white, eggshell, lemon yellow, mandarine, light brown, medium brown, olive, grey, blue, illusion, red, Gingiva, clear, clear high consistency h.c.)
  - CREATION VC Translucent, 2,4 ml (2 pieces high viscosity h.v., low viscosity l.v.)
  - CREATION VC Flow Dentine, 2 ml (13 shades, BW, A1-A3.5, B1-B3, HT, EN-L, EN-T)
  - CREATION VC Gingiva Opaque, 2,4 ml (4 shades, light pink, flamingo, rose, dark pink opaque)
  - CREATION VC Gingiva, 2,9 ml (4 shades, light pink, flamingo, rose, dark pink opaque)
  - CREATION VC Gingiva Modifier, 2,4 ml (6 shades, rose, dark pink opaque, violet, purple, light pink, dark pink)
  - CREATION VC Gingiva Translucent GT, 2,4 ml (1 shade, light pink)
- Accessories
  - CREATION VC Separator, 5 ml
  - CREATION VC Diamond Polisher, 8 g
  - CREATION VC Air Protect, 10 ml
  - CREATION VC Metal Primer, 5 ml
  - CREATION VC Composite Primer, 3 ml
  - CREATION VC Ceramic Primer, 3 ml
  - CREATION VC Disposable Palette, 10 pcs.
  - CREATION VC Protective Cover
  - CREATION VC Mixing Pad No. 22. 5 pcs.
  - CREATION VC Brush Round, Size 1, 10 pcs.
  - CREATION VC Brush Flat, Size 1, 10 pcs.
  - CREATION VC Brush Holder, 2 pcs.
  - CREATION VC Brush No. 7, 1 pc.
  - CREATION VC Syringe Plunger

### CAUTION

- Wear eye protection and plastic or rubber gloves during operation in particular to avoid direct contact with paste and air inhibited resin layers in order to prevent possible sensitivity.
- CREATION VC Metal Primer, CREATION VC Composite Primer, CREATION VC Air Protect and CREATION VC Separator are highly flammable. Do not use near open flame. Keep away from sources of ignition. Keep away from direct sunlight.
- When adjusting or polishing the resin, use a dust collector and wear a dust mask to avoid inhaling dust.
- In case of contact with oral tissues or skin, immediately remove with cotton. Flush with water.
- In case of contact with eyes, immediately flush with water and seek medical attention.
- Dispose of all waste according to local regulations.
- In rare cases the product may cause sensitivity to some people. If such reactions are experienced, discontinue use of the product and seek medical attention.
- Always wear personal protective equipment (PPE) such as gloves, face masks and safety eyewear.

\* Observe the manufacturer's instructions for use and make sure that these devices have the same light output as the GC LABOLIGHT LV-II, III and GC STEPLIGHT SL-I.

Some products referenced in this IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <https://www.creation-willigeller.com/en/media-center/manuals-brochures/>

Undesired effects - Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: [https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en) as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental). In this way you will contribute to improve the safety of this product.

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#### CONFEZIONI

- Assortimenti
  - CREATION VC Kit 1 (16 colori A1-D4), 54 siringhe con accessori
  - CREATION VC Kit 2 (6 colori A1-A3.5, B2, B3), 30 siringhe con accessori
  - CREATION VC Gingiva Kit (15 colori), 15 siringhe con accessori
  - CREATION VC Make up Neo (19 colori), 2 siringhe, 17 flaconi con accessori
  - CREATION VC kit accessori (contenuto: vedere 3. Accessori)
- Confezioni refill
  - CREATION VC Base Opaque, 2,4 ml (1 colore)
  - CREATION VC Opaque, 2,4 ml (16 colori, A1-D4)
  - CREATION VC Cervical Opaque, 2,4 ml (1 colore)
  - CREATION VC Opaque Dentine, 2,9 ml (5 colori: havanna, curry, orange, ivory, cuba)
  - CREATION VC Dentine, 2,9 ml (16 colori: A1-D4)
  - CREATION VC Enamel, 2,9 ml (4 colori)
  - CREATION VC Pearl Enamel, 2,9 ml (2 colori: blue, flamingo)
  - CREATION VC Effect Enamel, 2,9 ml (3 colori: light yellow, medium yellow, heavy yellow)
  - CREATION VC Transpa, 2,9 ml (4 colori: blue, white, yellow, grey)
  - CREATION VC Neck Transpa, 2,9 ml (3 colori: khaki, sand, honey)
  - CREATION VC Make up Neo, 2,6 ml (17 colori: MUN-A, MUN-B, MUN-C, white, eggshell, lemon yellow, mandarine, light brown, medium brown, olive, grey, blue, illusion, red, Gingiva, clear, clear high consistency h.c.)
  - CREATION VC Translucent, 2,4 ml (2 pz.: high viscosity h.v., low viscosity l.v.)
  - CREATION VC Flow Dentine, 2 ml (13 colori: BW, A1-A3.5, B1-B3, HT, EN-L, EN-T)
  - CREATION VC Gingiva Opaque, 2,4 ml (4 colori: light pink, flamingo, rose, dark pink opaque)
  - CREATION VC Gingiva, 2,9 ml (4 colori: light pink, flamingo, rose, dark pink opaque)
  - CREATION VC Gingiva Modifier, 2,4 ml (6 colori: rose, dark pink opaque, violet, purple, light pink, dark pink)
  - CREATION VC Gingiva Translucent GT, 2,4 ml (1 colore: light pink)
- Accessori
  - CREATION VC Separator, 5 ml
  - CREATION VC Diamond Polisher, 8 g
  - CREATION VC Air Protect, 10 ml
  - CREATION VC Metal Primer, 5 ml
  - CREATION VC Composite Primer, 3 ml
  - CREATION VC Ceramic Primer, 3 ml
  - CREATION VC Disposable Palette, 10 pz.
  - CREATION VC Protective Cover
  - CREATION VC Mixing Pad n. 22, 5 pz.
  - CREATION VC Brush Round, misura 1, 10 pz.
  - CREATION VC Brush Flat, misura 1, 10 pz.
  - CREATION VC Brush Holder, 2 pz.
  - CREATION VC Brush n. 7, 1 pz.
  - CREATION VC Syringe Plunger

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